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Introduction

The OptiFlow breast pump is designed to use anywhere you like. The pump is small and light so you can put it in your bra and take it with you wherever you go.

Hands-free

Convenient, without hoses or hanging bottles

Rechargeable battery

Including battery for up to 6 sessions

Memory function

Continue pumping with the last used settings

Closed system

Protects breast milk and pump from contamination

BPA-free

Made of BPA-free material and suitable for the dishwasher

Do you have questions or do you want to share feedback with us? We would love to hear from you, please send us an email at info@nutrea.nl

Table of contents


Read before use	15
The different parts	16
Assembling the breastpump	17
Explanation of the buttons	18
How to use the breast pump	19
Cleaning and disinfecting	22
Troubleshooting	23
Warranty policy	25
Product specifications	26

The product images shown are for illustrative purposes only and may not accurately represent the product

Read before use

- Switch off the breast pump before removing it from the breast
- Do not wash the breast pump motor in water or other liquids
- The parts must be dry before storage to prevent mold formation
- Contact your lactation consultant if you are not pumping enough milk and stop pumping if there is no milk flow and you do not get a let-down reflex
- Contact your lactation consultant if pumping is painful.
- Do not use the breast pump during pregnancy.
- Do not use aggressive and/or abrasive cleaning agents, as this may damage the parts
- Store in a cool, dry place and avoid direct sunlight
- Replace parts or the entire product if it shows signs of wear or damage
- Do not use a cleaning brush or brush, as this can damage parts and shorten their life
- Use the correct size breast shield or a breast shield reducer to pump comfortably
- Keep out of reach of children
- Only use the supplied adapter to prevent damage to the control module
- Wash your hands thoroughly before using the breast pump and dry all parts thoroughly to prevent water damage to the breast pump motor
- Do not use the breast pump if you have the impression that the packaging is damaged or has been opened before and contact customer service immediately

The different parts

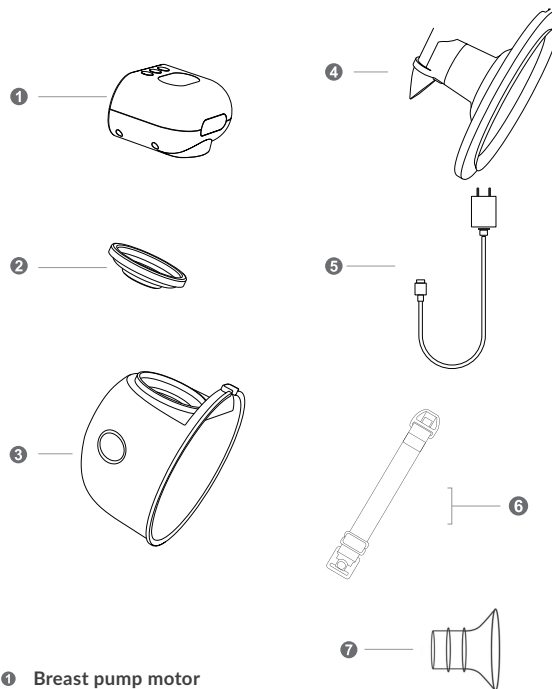1 Breast pump motor

2 Silicone diaphragm

3 Bottle 240ml

4 Breast shield

5 Adapter with USB cable

6 Bra strap extension

**7 Breast shield reducer
17 mm, 19 mm, 21 mm**

Assembling the breast pump

! Wash your hands thoroughly before assembling the pump. Before use, ensure that all washable parts are cleaned and disinfected as described in this manual. And make sure all parts are dry before assembling.

STEP 01

Place the silicone diaphragm on the bottle and press firmly


STEP 02

Place the breast shield in the bottle. Make sure the valve in the back of the bottle is tight before connecting the brim of the breast shell to the bottle

! If the breast shell and the bottle are properly connected, there will be a small opening at the top, which you can use to pour out the milk.

STEP 03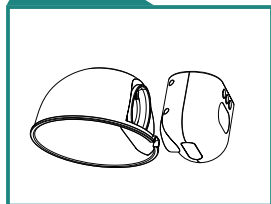


Place the breast pump motor on the bottle and press the motor firmly

STEP 04

The installation is complete and the breast pump is ready for use

Explanation of the buttons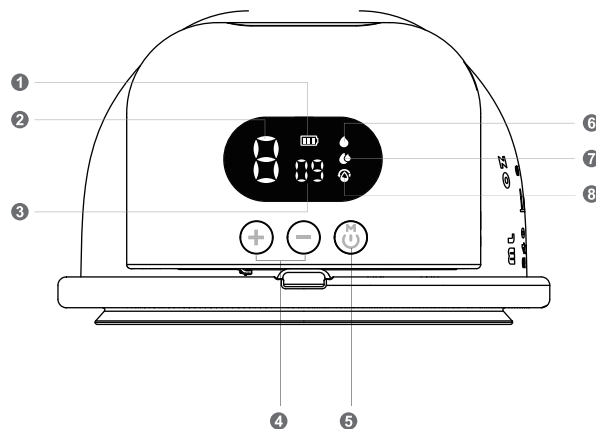

- | | |
|------------------------|-------------------------------|
| 1 Battery percentage | 5 On/off switch & Mode button |
| 2 Vacuum level display | 6 Massage mode |
| 3 Time display | 7 Expression mode |
| 4 Adjust vacuum level | 8 2-in-1 mode |

How to use the breast pump?

Instructions before use:

Make sure that all washable parts of the breast pump are disinfected and assembled according to the instructions in this manual. Wash your hands before use and massage the breast briefly. Then sit up straight and align the center of the silicone breast shield with your nipple. Press the breast shield firmly against the breast to create a vacuum.

On/off switch

Press this button to activate the motor. The motor switches off automatically after 30 minutes.


Adjust vacuum level

With these two buttons, you can adjust the vacuum level. Each mode has 9 vacuum level settings. This allows you to set the intensity that feels most comfortable for you and is the most efficient. The intensity is indicated in the center of the display by a number from 1 to 9.


Mode button

There are three different working modes, namely massage mode, expression mode and 2-in-1 mode. When activating the control module, the default working mode is intensity 1 of the massage mode. Press the button to switch to pumping mode and then to switch to 2-in-1 mode. The massage mode is a faster pumping rhythm to stimulate the ejection of the milk. The expression mode is a long and deep pumping rhythm to simulate a baby's drinking. The 2-in-1 mode is two short and one deep pumping rhythm. You use this mode when the breast is almost empty and especially when the milk flow is slow.

Placement in the bra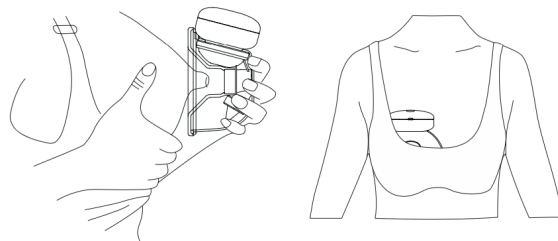

Switch off the pump before removing it from the breast

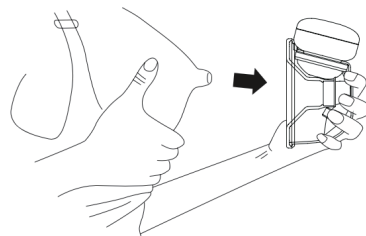Pump positions

✓ Ideal

Upright

✓ Good

Slightly bent over

✓ Fine

Bent over

✗ May leak

Sit back

✗ Does not work

back or side position

Cleaning and disinfecting

Refer to the following methods for disinfecting the washable parts.

Remove milk residues

Milk residues can be removed in one of the following ways:

- ① Rinse with clear, cold water
- ② Wash off with warm water
- ③ On the top rack of the dishwasher

Sterilize:

The washable parts can be sterilized in one of the following ways:

- ① Cook in boiling water for about 5 minutes
- ② Sterilization in microwave or steam device. We do not recommend using this method for more than 5 minutes

⚠ The motor, adapter and USB cable cannot be disinfected or cleaned with water

⚠ Some microwave or steam sterilizers can reach high temperatures that can damage plastic parts

⚠ Dry all parts thoroughly before use

Troubleshooting

⚠ There is no suction or very little suction

- ❶ Check that all parts are clean and dry.
- ❷ Check that parts such as the valve are not damaged.
- ❸ Check that all parts are properly connected.
- ❹ Make sure that the breast shield seals the breast airtight all around.

⚠ The breast pump motor no longer turns on

- ❶ Check if the battery is fully charged
- ❷ Check whether the adapter and charging cable are properly connected
- ❸ Contact Nutrea customer service

⚠ I have little or no production


- ❶ Is the breast pump too tight in the bra or too loose? Otherwise, try pumping without a bra to find the right pressure to rule out this problem.
- ❷ What size shield do you use? Did you measure this correctly using our size chart?

We recommend starting pumping in massage mode. The program starts at level 1 and automatically switches to level 2 after 45 seconds. Automatic switching takes place up to and including level 4, after which you can manually switch to a higher level. This position is used to trigger the let-down reflex. Only when the milk starts to flow do you switch to the next program, the expression mode. In expression mode, you can find the right suction power and also keep an eye on the milk flow. When the breast is almost empty, you switch to the last mode (2-in-1).

⚠ All parts must be dry before use to prevent damage to the breast pump motor

⚠ Do you have pain while pumping? Please contact your lactation consultant or nurse immediately

Sit upright while pumping. Do not lean forward or lie on your side. See also the pumping positions in the previous chapter.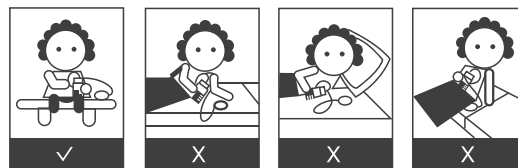

Warranty policy

We offer a 24 month warranty on the pump mechanism (control module) and a 90 day warranty on the washable parts from the original date of purchase. This warranty is provided free of charge and is without prejudice to any rights granted under consumer laws or regulations in the country of purchase. It does not exclude, limit or affect any consumer rights or legal claims arising from national laws governing the sale of consumer products.

The warranty does not cover normal wear and tear or damage caused by use, physical or natural destruction, accident, abuse, neglect or other external causes. Any attempt to disassemble the pump mechanism (control module) will void this warranty.

In the event that Nutrea decides to replace the product or parts, the term of this warranty will not be extended for the replacement product.

Use by more than one person may create health hazards and void the warranty.

Keep your proof of purchase in case you need to contact customer service.

Product specifications

Brand:	Nutrea
Model:	OptiFlow
EAN number:	8720648346705
Bottle capacity:	240 ml
Average charging time:	120 minutes
Average usage time:	90 minutes
Vacuum range:	40 to 280 mmHg
Power connection:	5V DC
Battery type:	Lithium Polymer
Power:	1100 mAh
Dimensions:	139*70*87 mm
Weight:	250 gram
USB connection:	USB C